**NOTATKA DORADCZA
ADVISORY NOTE**

Data wydania Date of issue	2020-05-19
Numer notatki Note number	ND_2005_01
Nazwa i numer katalogowy wyrobu Product name and catalogue number	ACCENT-200 LDH Cat. No 7-239
Numer serii Lot number	all lots
Data ważności Expiry date	n/a
Opis sytuacji Issue description	
For determinations where the reaction direction is negative and the substrate depletion value is equal to the maximum absorbance measured by the analyzer, problems with the calibration of the test may occur. In order to prevent such a situation, remove the substrate depletion value according to the application attached to the note.	
Konieczne działania Necessary actions	
Read carefully advisory note, complete and return the verification form. Update the application on ACCENT S120, ACCENT MC240 or ACCENT M320 analyzer before use.	

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Please provide this information to all Users of the above product and keep the Advisory Note as a quality system documentation.

Please complete and return the attached Verification Form to confirm receipt of the Advisory Note.

We apologize for any inconvenience caused by the above situation.

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X *Anna Smolira*