

ADVISORY NOTE

Date of issue	06.04.2022	
Note number	ND_2204_03	
Product name and catalogue numer	Name	Cat. No
	Liquick Cor-GLUCOSE mini	2-218
	Liquick Cor-GLUCOSE 30	2-219
	Liquick Cor-GLUCOSE 60	2-201
	Liquick Cor-GLUCOSE 120	2-202
	Liquick Cor-GLUCOSE 500	2-203
	HC-GLUCOSE	4-501
	OS-GLUCOSE	9-401
	B50-GLUCOSE	5-510
	BIOLIS 50i GLUCOSE	5-510
	A-400 GLUCOSE	7-401
	A-800 GLUCOSE	7-801
	PRESTIGE 24i LQ GLUCOSE	4-201
	PRESTIGE 24i LQ GLUCOSE	4-401
	BIOLIS 15i GLUCOSE	4-708
	Liquick Cor-GLUCOSE BULK	2-275_R1
Lot number	all lots	
Expiry date	n/a	
Issue description		
<p>In order to ensure traceability, based on the primary standard: NIST Standard Reference Material 965b, values assigned for glucose in calibrators and control materials were re-determined:</p> <ul style="list-style-type: none"> - CORMAY SERUM HN (Cat. No 5-172) lot HN09, - CORMAY SERUM HP (Cat. No 5-173) lot HP09, - CORMAY MULTICALIBRATOR LEVEL 1 (Cat. No 5-174 / 5-176) lot XXX-03X, - CORMAY MULTICALIBRATOR LEVEL 2 (Cat. No 5-175 / 5-177) lot XXX-03XX and XXX-04X. <p>The new values have been placed in Erratum NDE_2204_01 (Attachment 1).</p>		
Necessary actions		
<ul style="list-style-type: none"> • read carefully Advisory Note, Erratum NDE_2204_01 (Attachment 1) and use the re-determined values of calibrators and controls during determinations of glucose, • perform calibration and quality control prior to ordering patient samples, • complete and return the Verification Form. <p>The note is only for reference. Do not discard the product!</p>		

Please provide this information to all Users of the above product and keep the Advisory Note as a quality system documentation.

Please complete and return the attached Verification Form to confirm receipt of the Advisory Note.

We apologize for any inconvenience caused by the above situation.

Sincerely,

X *Anna Smolira*

Anna Smolira
 Quality Assurance
 Podpisany przez: Anna Smolira