

ADVISORY NOTE

Date of issue	29 th March 2023	
Note number	ND_2303_03	
Product name and catalogue numer	Name	Cat. No
	Liquick Cor-GLUCOSE 30	2-219
	Liquick Cor-GLUCOSE 60	2-201
	Liquick Cor-GLUCOSE 120	2-202
Lot number	all lots	
Expiry date	n/a	
Issue description		
Results obtained during routine reagents quality control exceeded the internal acceptance criteria. To solve the problem:		
1. The value of the 2-STANDARD , which is included in Liquick Cor-GLUCOSE 30 (Cat. No. 2-219) and Liquick Cor-GLUCOSE 60 (Nr kat. 2-201) kits, was redetermined, based on the primary standard NIST 965b. New standard value is:		
REAGENT	LOT NUMBER	STANDARD VALUE
2-STANDARD	XXX-XX58	91.5 mg/dl (5.08 mmol/l)
	XXX-XX59	91.1 mg/dl (5.06 mmol/l)
	XXX-XX60	94.3 mg/dl (5.23 mmol/l)
2. Due to the changed values of the aqueous standards, values assigned for glucose for manual method in control materials:		
<ul style="list-style-type: none"> • CORMAY SERUM HN (Cat. No 5-172) lot HN10, • CORMAY SERUM HP (Cat. No 5-173) lot HP10, were also re-determined and reported in the erratum NDE_2303_02 (Attachment 1).		
Necessary actions		
<ul style="list-style-type: none"> • Read carefully Advisory Note, Erratum NDE_2303_02 (Attachment 1) and use the re-determined values of standard and controls during determinations of glucose. • Perform calibration and quality control prior to ordering patient samples. • Complete and return the Verification Form. 		
The note is only for reference. Do not discard the product!		

Please provide this information to all Users of the above product and keep the Advisory Note as a quality system documentation.

Please complete and return the attached Verification Form to confirm receipt of the Advisory Note.

We apologize for any inconvenience caused by the above situation.

Sincerely,