

MYTHIC 70 in-line filter tubing update

In order to improve functionality of MYTHIC 70, device fluidic system has been modified. This update adds Luer connector next to the filter so it will be easier for the customer to remove and clean the filter.

Tools needed:

In-line filter update kit, flat head screwdriver, torx 20 screwdriver, disposable gloves.



Installation:

Turn off the analyzer.


Open right side doors and bottom cover. Make sure that cable located in the back is disconnected.




Remove two screws which hold pump in place, remove pump from the noise dumping foam.

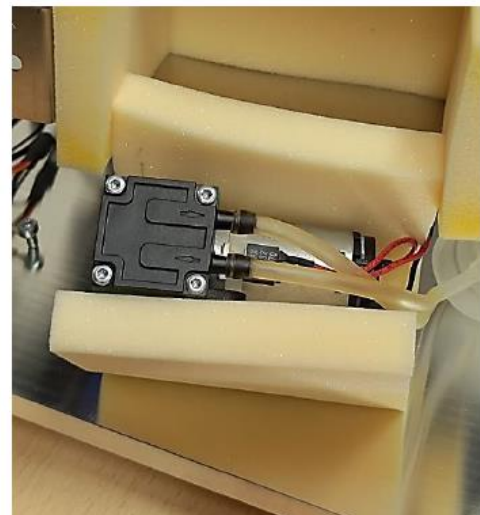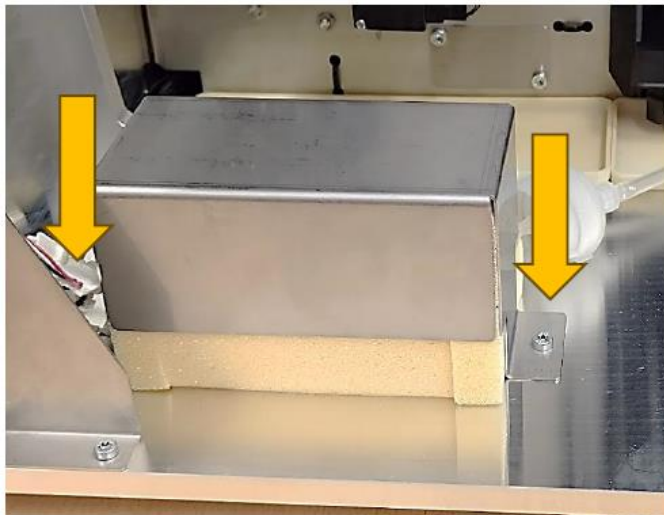



Remove the tube which connects in-line filter with input port of the pump (highlighted in red).

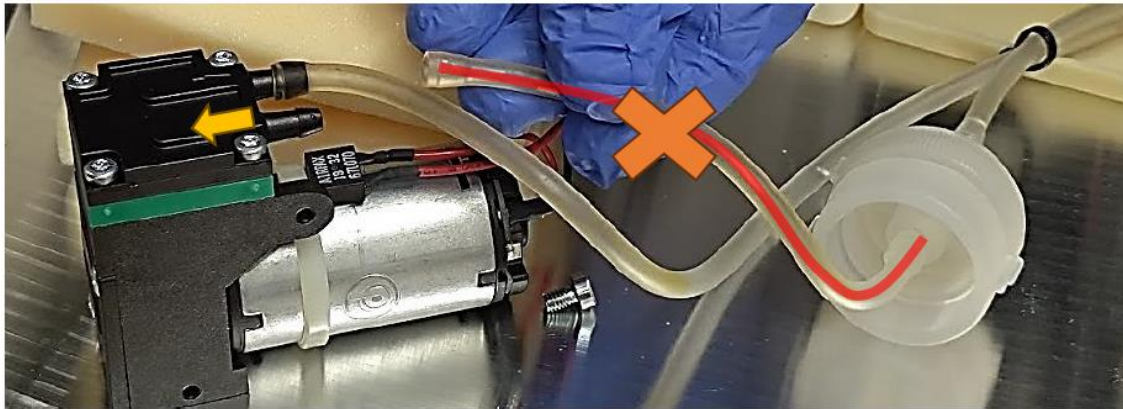

Install new tube (highlighted in green). Make sure that Luer connector is located next to the in-line filter.

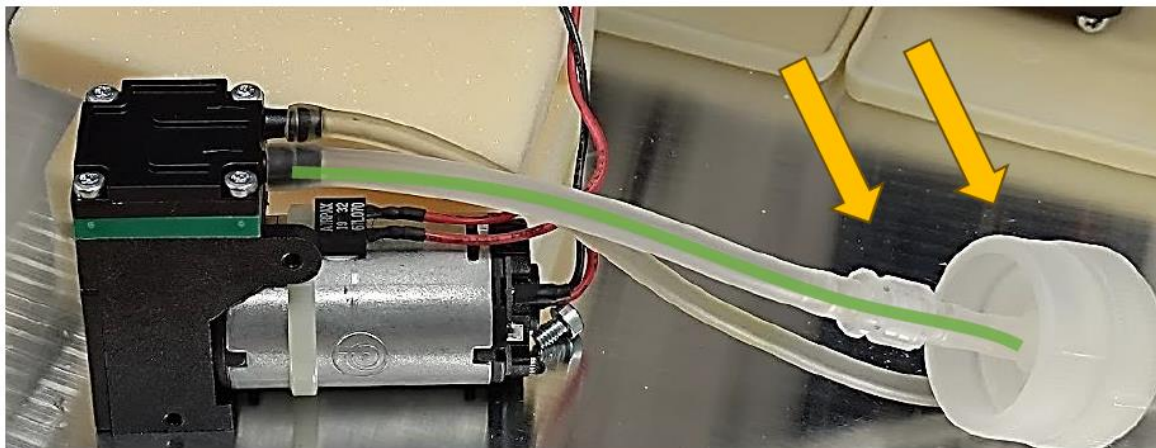

Assemble everything in reverse order. Place tubes and cables in correct locations. Make sure that grounding wire (left screw of pump's cover) is installed. Put all covers back.

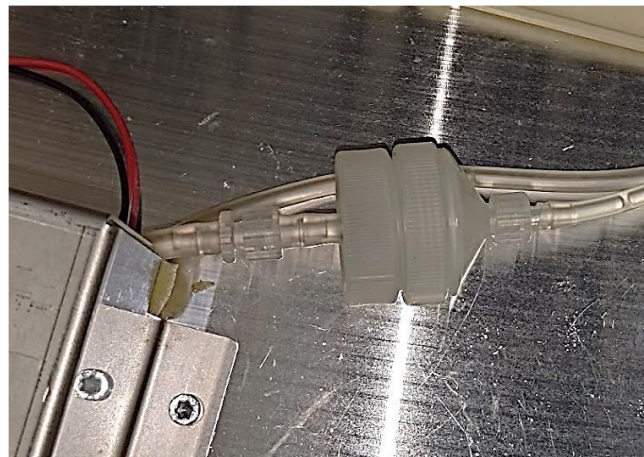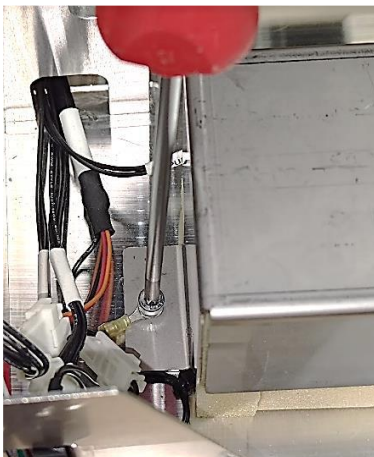

Device is now fully functional.