

**ADVISORY NOTE**

Date of issue	11.04.2024		
Note number	<b>ND_2404_01</b>		
Product name and catalogue number	MYTHIC 70 HEMATOLOGY ANALYZER		
Serial numbers	710521-000011	710223-000032	710423-000043
	710922-000020	710223-000033	710423-000044
	710922-000021	710423-000034	710423-000045
	710223-000024	710423-000035	710523-000047
	710223-000025	710423-000036	710523-000048
	710223-000026	710423-000037	710523-000049
	710223-000027	710423-000038	710523-000050
	710223-000028	710423-000039	710523-000051
	710223-000029	710423-000040	710523-000052
	710223-000030	710423-000041	710523-000053
	710223-000031	710423-000042	
Expiry date	N/A		
<b>Issue description</b>			
<p>MYTHIC 70 analyser has fluidic system consisted of directly connected filter and pump.</p> <p>In order to improve functionality of MYTHIC 70, device fluidic system has been modified. This update adds Luer connector next to the filter so it will be easier for the user to remove and clean the filter.</p> <p>The In-line filter update kit will be sent to you immediately. The method of update is described in attached instruction (Attachment No 1).</p>			
<b>Necessary actions</b>			
<ul style="list-style-type: none"> <li>• Read carefully the Advisory Note and the Attachment No 1.</li> <li>• Use received In-line filter update kit according to the attached instruction (Attachment No 1).</li> <li>• Complete and return the verification form.</li> </ul>			

**Please provide this information to all Users of the above product and keep the Advisory Note as a quality system documentation.**

**Please complete and return the attached Verification Form to confirm it of the Advisory Note.**

**We apologize for any inconvenience caused by the above situation.**

**Sincerely,**

Quality Assurance Manager