


New software updates

Are available on our website:

<https://cormaydiagnostics.com/>

1. Login to your account by pressing here (access is granted by your Sales Manager):

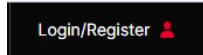


Type your username and password

Log in


E-mail

Password


I'm not a robot  reCAPTCHA
Privacy - Terms

Log in >


2. Go to Customer Support > Service Support > Service Documents section:

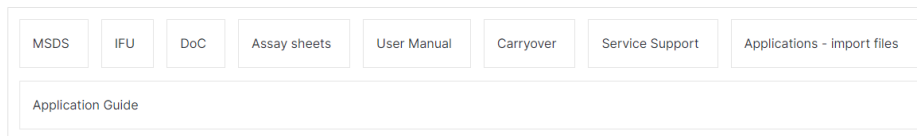


3. Go to section dedicated for analyzer for which you want to download software:

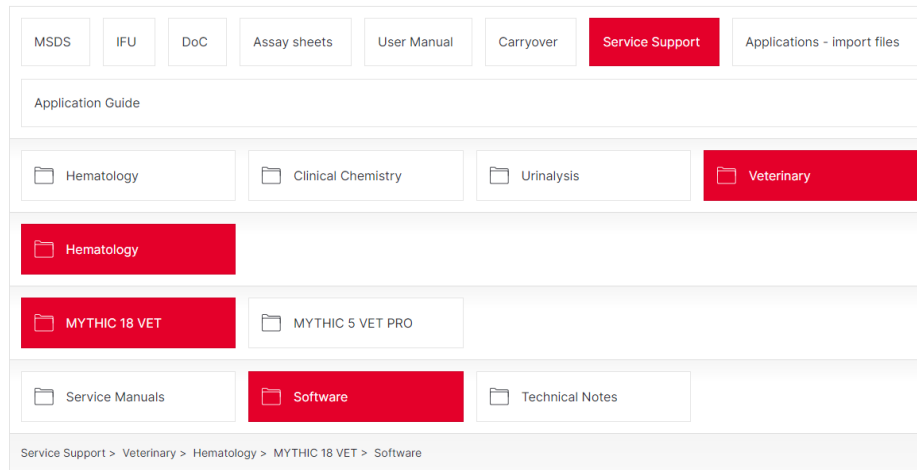

4. Download update file to your computer:

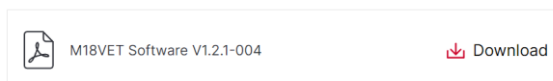

Downloaded file is a M18VET_V1.2.1_004.zip archive. Inside you can found update files:

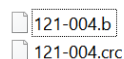

Files should be copied in to \M18VET\UPDATE\VERSION folder of your USB drive.